ZENDURA

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Instructions for Use

Intended Use

Zendura Thermoformable Materials are intended for the fabrication of orthodontic and dental appliances designed and planned by a registered orthodontist or registered dental lab technician.

Indications for Use

Zendura Thermoformable Materials allow the fabrication of orthodontic and dental appliances such as aligners and retainers.

Zendura AT and TRAK Thermoformable Materials allow the fabrication of attachment templates.

	Contraindications This material is contraindicated for patients with a history of allergic reaction to plastics.
	Warning: Keep Zendura thermoformable material away from excessive heat or ignition source. Provide suitable exhaust ventilation when thermoforming or trimming, as it may potentially form combustible dust.
	Warning: Excessive heating time during thermoforming will cause plastic to sag with possible risk of injury to operator.
	Caution: Zendura is provided pre-dried in moisture barrier packaging and should be thermoformed within 15 minutes of opening its protective foil pouch. This time can be reduced in a humid environment. Delayed thermoforming may cause bubbling and other defects that may render the final dental appliance unsuitable for use.
	Caution: Failure to properly prepare the model, thermoform, trim, or clean the appliance, or improper handling in general, may result in defects, cracking, or breakage that may create sharp edges that could cause injury, loose pieces that could be aspirated, and/or render the dental appliance unsuitable for use.
	Caution: Store unopened Zendura packages in a cool and dry place in their original packaging. Do not use Zendura thermoformable material if the moisture barrier bag sealing is damaged or compromised. Do not use Zendura thermoformable material after the expiration date indicated on the packaging. Product performance may be affected. Dispose of material and used dental appliances in accordance with applicable local regulations. Used dental appliances may be considered biohazardous waste.
	Caution: Zendura thermoformable material is intended for single use only. Reuse or repeated thermoforming may compromise the material's mechanical properties and device performance.

Important: This product is for use by a professional user, according to the instructions for use.

Important: Users and/or patients should report serious incident with medical device information to the manufacturer and to the national competent authority.

*U.S. Pat# 10,870,263 & 10,549,511; International patents pending. ZENDURA is a registered trademark of Bay Materials, LLC.

© Bay Materials, LLC. All rights reserved.

Thermoforming

Use of Roll format materials requires machinery and methods that are not available in most clinical environments. The purchaser of roll format products takes responsibility for developing thermoforming parameters and techniques that meet requirements for the prescribed dental appliance.

Cleaning

Wash Zendura appliance with mild soap and rinse well with water. Do not use hot water, vinegar, organic solvents or alcohol.

Storage and Disposal

Store unopened and sealed Zendura thermoformable material in a cool and dry place in their original packaging. Disposal should be handled in an environmentally sustainable manner according to local regulations. Used dental appliances may be considered a biohazard. Follow your country-specific laws, directives, standards and guidelines for the disposal of used devices.

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AR Experts B.V.
Etkon GmbH
UKCA Experts Ltd.
Institut Straumann AG
Emergo Asia Pacific Pty Ltd